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www.bionexia.de

## INTENDED USE

The BioNexia® CRP ratio Test is a semiquantitative rapid test for the detection and the monitoring of CRP levels in whole blood samples. The cut-off of detection of the test is 10 µg/ml.

The test is intended for professional in vitro diagnostic use only.

## SUMMARY

CRP (C-reactive protein) is an acute-phase-protein. It is a classical marker that can be used to detect an inflammation reaction in the body. CRP is synthesized in the liver and is induced by proinflammatory cytokines, especially Interleucin-6. Via the plasma, CRP is delivered to the sites of inflammation, where it actively contributes to the innate immune response.

Systemic inflammation is usually accompanied by an increase of CRP concentration in blood. Especially bacterial infections, sterile tissue damage e.g. after surgical procedures, trauma or extreme physical stress, tumors especially in the metastasing stage, some autoimmune diseases as well as malignant systemic diseases (Hodgkin's and Non-Hodgkin's Lymphoma) can induce a CRP increase.

For diagnostic purposes, CRP is often used to differentiate between viral and bacterial infections, as the increase in CRP is more enhanced in bacterial infections. CRP is also frequently used to monitor the therapeutic effect of e.g. an antibiotic treatment.

The BioNexia® CRP ratio Test is an immunochromatographic test, based on two specific antibodies against human CRP. The concentration-dependent formation of test lines allows a rapid semiquantitative determination of CRP in whole blood samples.

## TEST PRINCIPLE

The tip of the test strip is dipped into the test sample which is diluted with buffer solution. The sample now moves through the test strip from bottom to top. If the test sample contains CRP, it attaches to the first anti-CRP antibody which is conjugated with a red gold colloidal for color marking. The red CRP-antibody-gold complex, together with the sample liquid, diffuses through the membrane that is pre dispensed with lines of different amounts of the second

Deutsch

English

Français

Español

Italiano

Polska

anti-CRP antibody. The CRP-antibody-gold complex is immobilized by the antibodies coated on the membrane leading to the formation of red lines. The number of lines depends on the CRP concentration in the sample. The more CRP is contained in the sample, the more red lines become visible.

At the end of the membrane, a red control line is formed, which indicates that the test has performed properly. The formation of the control line is independent of the CRP concentration in the sample. Absence of the red control line indicates an invalid result. The test should then be repeated.

## STORAGE

The BioNexia® CRP ratio Test should be stored at room temperature (15-30 °C). If the test is stored cool, salts in the buffer solution can flocculate. However, usually this has no effect on the quality of the product.

Unopened, each component can be used until the expiration date.

## PRECAUTIONS

- For in vitro use only
- For professional use only
- For single use only
- Do not use beyond expiration date
- Do not use the test if the plastic foil packaging is damaged
- Store and transport the test always at the stated temperatures
- Blood samples may be potentially infectious. Therefore appropriate precautions should be applied for their handling and proper disposal
- The sample buffer contains low concentrations of the preservative sodium azide below the exemption limit of 0.1 %
- Please keep to the instructions given in this package leaflet and keep exactly to the given sample volume and the reading time

## TEST MATERIALS SUPPLIED

The test kit contains reagents and materials for the performance of CRP rapid tests. For the exact number, please refer to the label on the carton. The kit has the following components:

- individually sealed tests strips in aluminum covers
- plastic tubes containing buffer solution
- End-to-end capillaries (10 µl)
- sterile lancet devices (under the Directive 93/42/EEC included as additional material, CE<sub>0413</sub>)

- racks for the plastic tubes
- 1 instruction sheet

## REQUIRED MATERIALS

- Stop watch or timer

## SPECIMEN COLLECTION

### Preparation

Before performing the test, please make sure that all components are brought to room temperature (see storage). Cold buffer solution or moisture condensation on the test cassette can lead to invalid test results.

1. Take a vial with buffer solution out of the kit. Document patients name or ID. Open the screw cap.

### Blood Sample Taking

Use the supplied lancet device according to the following instruction to extract a drop of blood from the finger tip:

2. Disinfect the finger tip.
3. Uncover the needle of the lancet device by turning and pulling-off the protecting cap.
4. Press the lancet device onto the disinfected finger tip and release the shooting mechanism by pushing the circular button.
5. With the supplied capillary, take a volume of 10  $\mu\text{l}$  from the blood drop. It is important that the end-to-end capillary is filled until the upper end. Due to hygienic reasons, hold the capillary with a capillary holder or tweezers. Alternatively, the blood can also be taken with a micro pipette. Please note: In case of using micro pipettes or other capillaries a sample volume of exactly 10  $\mu\text{l}$  must be administered. Please dilute the blood sample immediately to avoid clotting.

### Sample Dilution / Sample Stability

6. Administer the blood-filled end-to-end capillary into the accordingly prepared reaction tube with dilution buffer. Alternatively, the 10  $\mu\text{l}$  of blood can be added directly with the micro pipette into the buffer.

7. Close the vial and shake the sample by hand forcefully for approximately 10 seconds so that the blood is released from the capillary and sample and dilution buffer mix themselves well.
8. Let the diluted sample rest for approximately 1 minute.
9. The sample can then be used immediately or stored for up to 8 hours.

### Note

EDTA-, citrate- or heparin- blood can be used as well. Before performing the test, it has to be diluted accordingly with the supplied buffer.

## TESTING PROCEDURE

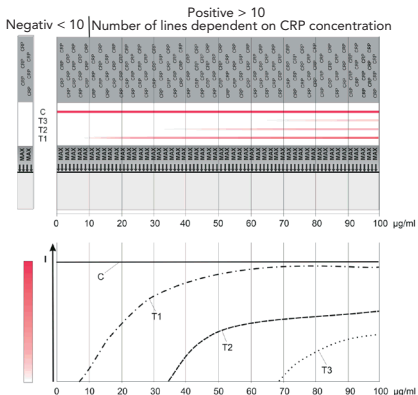
After opening the aluminum covers the tests should be performed quickly, as they are sensitive to humidity.

1. Open the aluminum cover and take out the test strip at the end labelled with CRP. Do not touch the white result field with your fingers.
2. Open the vial with the diluted sample and put the test strip with the other end into the liquid. Make sure to dip the test strip not above the maximum marking line (MAX). Avoid in any case a direct wetting of the result field by lateral dashes or dipping the test strip to deep.
3. Leave the test strip for min. 10 seconds in the diluted sample until the slightly pink-colored liquid front becomes visible in the result field.
4. Take out the test strip and put it to a plane underlay that does not dehydrate the test strip (e.g. the test pouch). Alternatively, the test strip can remain in the tube. Start the stop watch.
5. Wait for the appearance of colored lines. Read the result after 5 minutes. Please stick exactly to this time line to ensure correct semiquantitative results.

## INTERPRETATION

(please note that the test must be read after exactly 5 minutes)

To interpret the results please look at the line pattern that has formed in the white membrane field.

### Positive Result:

If more than one line (control line C) appears below the holder, the test result has to be regarded as positive, that is CRP was detected in the sample.

The formation of the test result lines (T1, T2 and T3) is dependent on the concentration. The more lines are visible, the higher is the CRP concentration in the sample.

- 1 red test line (T1) and 1 red control line (C):  
AT CRP concentrations of approximately 10 µg/ml and higher a test line that is faint in the beginning appears in the lower area in addition to the control line.  
With increasing CRP concentration the color intensity of the line is enhanced.
- 2 red test lines (T1 and T2) and 1 red control line (C):

From approximately 40  $\mu\text{g/ml}$  on a second faint colored test line appears above the first test line.

- 3 red test lines (T1, T2 and T3) and 1 red control line (C):  
From approximately 80  $\mu\text{g/ml}$  on a third faint test line appears above the first two test lines.

The diagram above shows the formation of the test lines depending on the CRP concentration. You can see how the number of the test lines increases with increasing CRP concentration. The color intensity is faint when the lines begin to appear and becomes enhanced with increasing CRP concentration.

Please note that the color intensity of the three lines is different. You can see that in the shown concentration range the lowest line is most intensively colored. The middle and especially the upper test line are significantly less colored.

In contrast to the color intensity of the test lines, the color intensity of the control line is independent of the CRP concentration and constantly shows a deep red color.

#### **Negative Result:**

Only the red control line appears. The CRP concentration in the sample is  $<10 \mu\text{g/ml}$ .

Semiquantitative rapid test for the detection of C-reactive protein (CrP) in human whole blood

#### **Invalid:**

The deep red colored control line (C) in the upper area of the membrane field is not visible. In this case, the test is invalid, even if the test line(s) is / are visible. The test should be repeated.

#### **Note**

Please read the test only once after 5 minutes. It is quite natural for immunochromatography and the kinetics of rapid tests, that the intensity of all lines increases over time. If the test is read later, an accurate semiquantitative interpretation of the test result is not ensured any more, as low concentrations would be estimated too high.

#### **EXPECTED RESULTS**

CRP plasma levels increase within 6 to 8 hours after occurrence of an acute event like for example a bacterial infection or trauma and reach their peak

within approximately 48 hours after the occurrence of an event. The levels fall quickly after the causing event stops, with the given CRP half-life being 48 hours.

Usually, the severity of the inflammation and the inflammation activity influence the extent of the CRP increase. Values of 10 to 40 µg/ml often coincide with mild inflammation like local bacterial infections, abscess, mild trauma, malignant tumors, most viral diseases etc. Up to 100 µg/ml indicate severe illness with inflammation that usually requires immediate medical treatment measures.

Values higher than 100 µg/ml are found e.g. in bacterial sepsis or major surgical procedures (Thomas, 2008).

### LIMITATIONS OF THE TEST

- CRP is not a specific marker for a certain disease. As with all in vitro diagnostics, the result should not be interpreted on its own but must be correlated with clinical findings. Often, CRP increase occurs before the symptoms become apparent, therefore temporal connections should be considered as well.
- Intraindividual variation of CRP values is relatively high. In general, values > 10 µg/ml can be regarded as elevated in the majority of the patients.
- The test is not suitable for risk stratification of coronary heart diseases. For this, it is not sensitive enough.

### ANALYTICAL PERFORMANCE

The test was calibrated against the international human CRP reference standard of the WHO 85/506. The detection limit of the BioNexia® CRP Test is adjusted in such a way that a concentration of 10 µg/ml in the undiluted specimen material after dilution with the supplied buffer yields a positive red line.

If the undiluted specimen contains more than 40 µg/ml CRP, a second red line appears. With concentrations higher than 80 µg/ml CRP, a third red line appears. These concentration ranges could be confirmed in comparison to a quantitative reference test.

Hook effects could not be proven up to a concentration of 2,000 µg/ml CRP. At this concentration, the test indicated results of >80 µg/ml.

## QUALITY CONTROL

Good laboratory practice (GLP) recommends the use of control materials to proof proper test kit performance.

## BIBLIOGRAPHY

1. Macy EM, Hayes TE & Tracy RP (1997): Variability in the measurement of C-reactive Protein in healthy subjects; implications for reference intervals and epidemiological applications. Clin. Chem. 43, 52-58.
2. Thomas L, Labor und Diagnose, 7. Auflage (2008), TH- Books Verlagsgesellschaft mbH, Frankfurt/Main

## SYMBOLS:

	For in vitro diagnostic use only
	For single use only
	Content
	Expiration date
	Lot number
	Storage temperature

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